




MAESTRO ORTHO STUDIO 32 & 64 BIT INSTALL INSTRUCTIONS

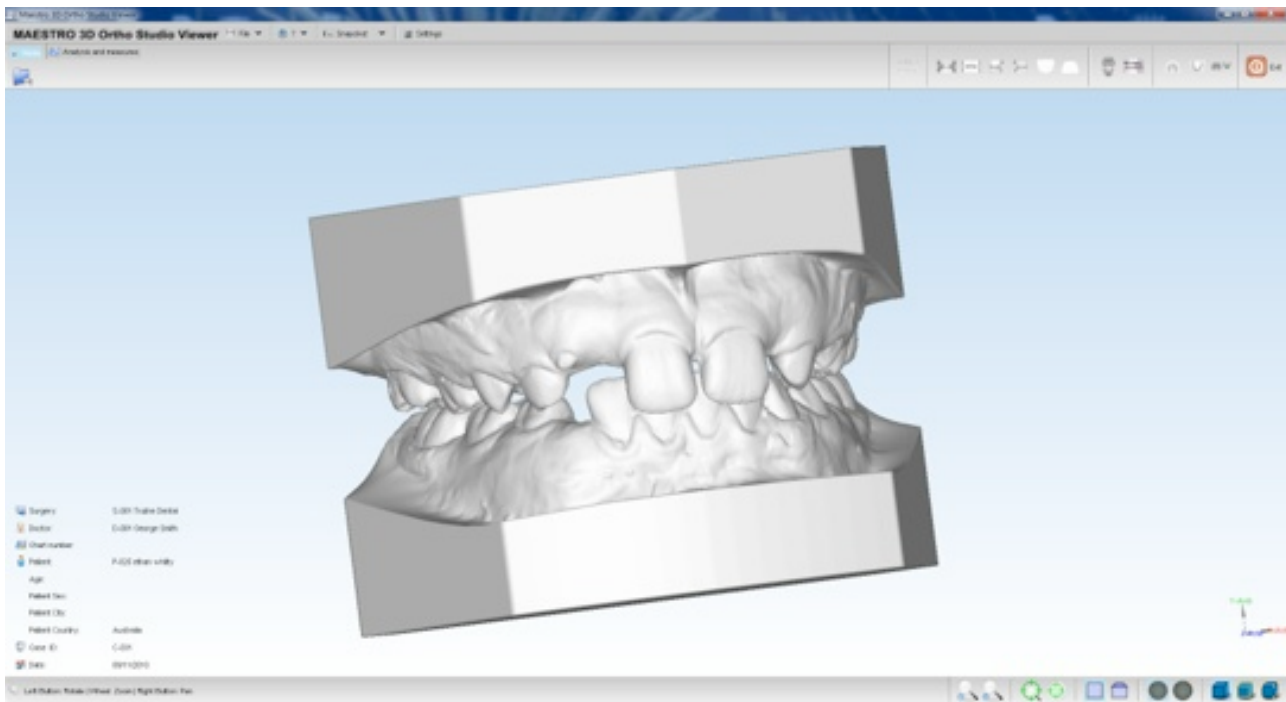

System Requirements

Windows 7 with a suitable Graphics Card

32 Bit for 32 Bit Version

64 Bit For 64 Bit Version (supports virtual set-ups)

Most modern computers are supported.




1. Double click the installer program and select your language




2. The welcome screen will appear
...click "NEXT"



3. The license screen will appear
...click "NEXT"

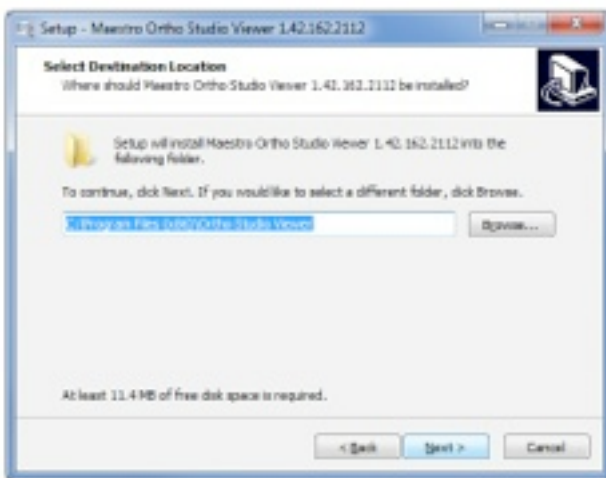


4. Destination folder is created
...click "NEXT"

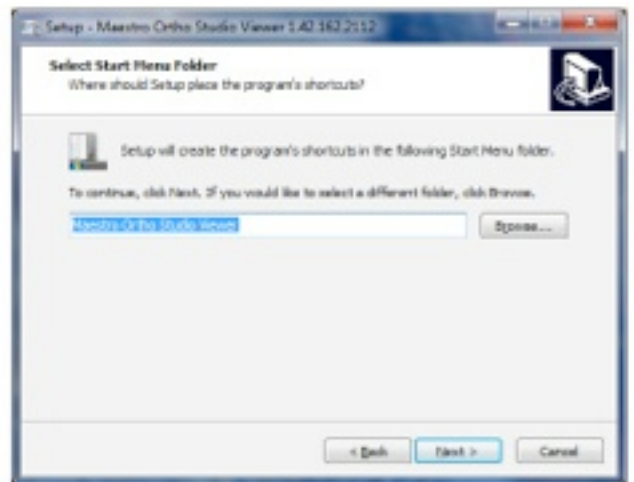

5. The program will create a shortcut
in the start menu
...click "NEXT"




6. The program will create a shortcut
and quick launch icons if you wish
...click "NEXT"

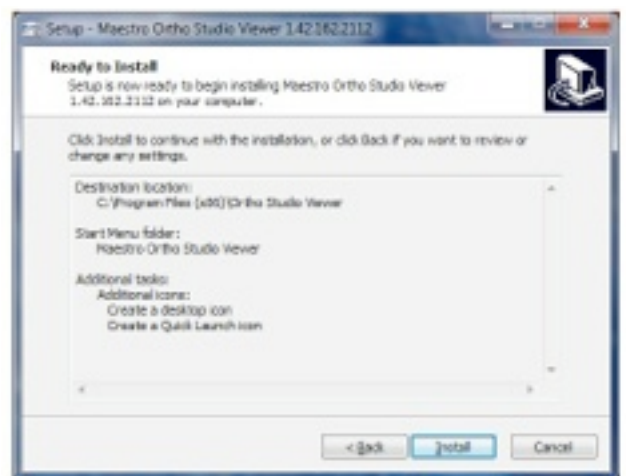

7. This just gives a summary
...click "NEXT"



8. When the program installs the first time you will be presented with this window.

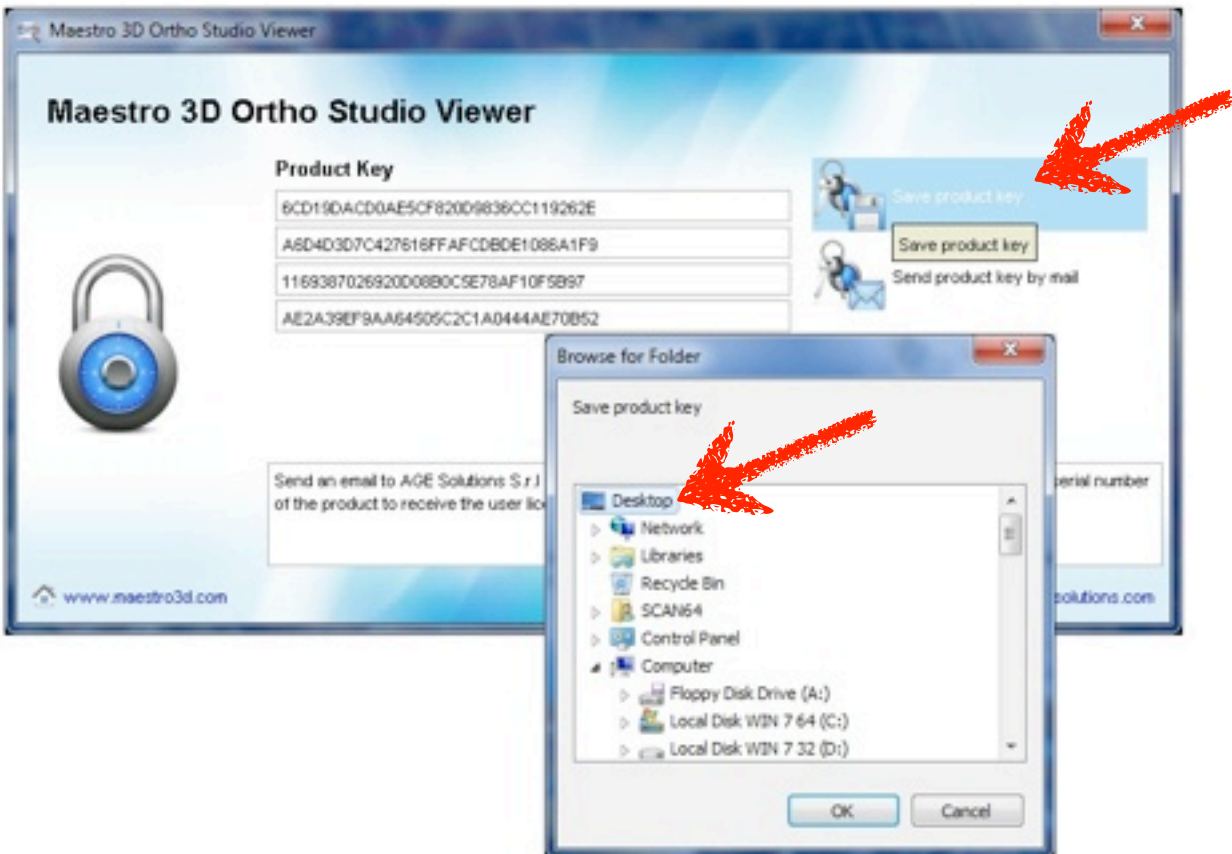


9. Click on the “Save Product Key” and a window will appear asking you to browse to folder where you can select a place to save the key. It is suggested you save it to your desktop so you can find it easily.



10. Here is the license key saved to the desktop. (see red arrow) Email the file to info@trulinedental.com.au and request a license file.




11. Alternatively you can click the “Send product key by mail” and it will open a mail file automatically with the key file attached but only if you are using Outlook for your mail program.

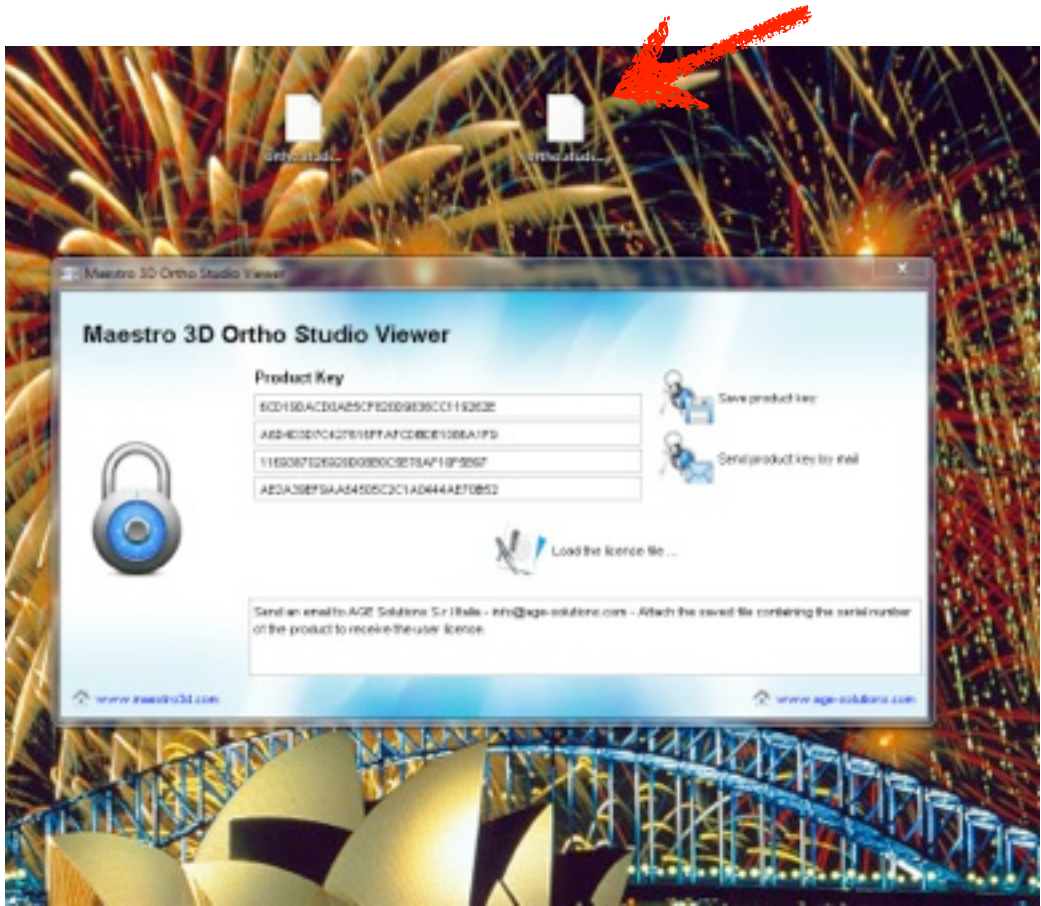


12. Once you have your license file save it to your desktop.




13. Click the load license file button

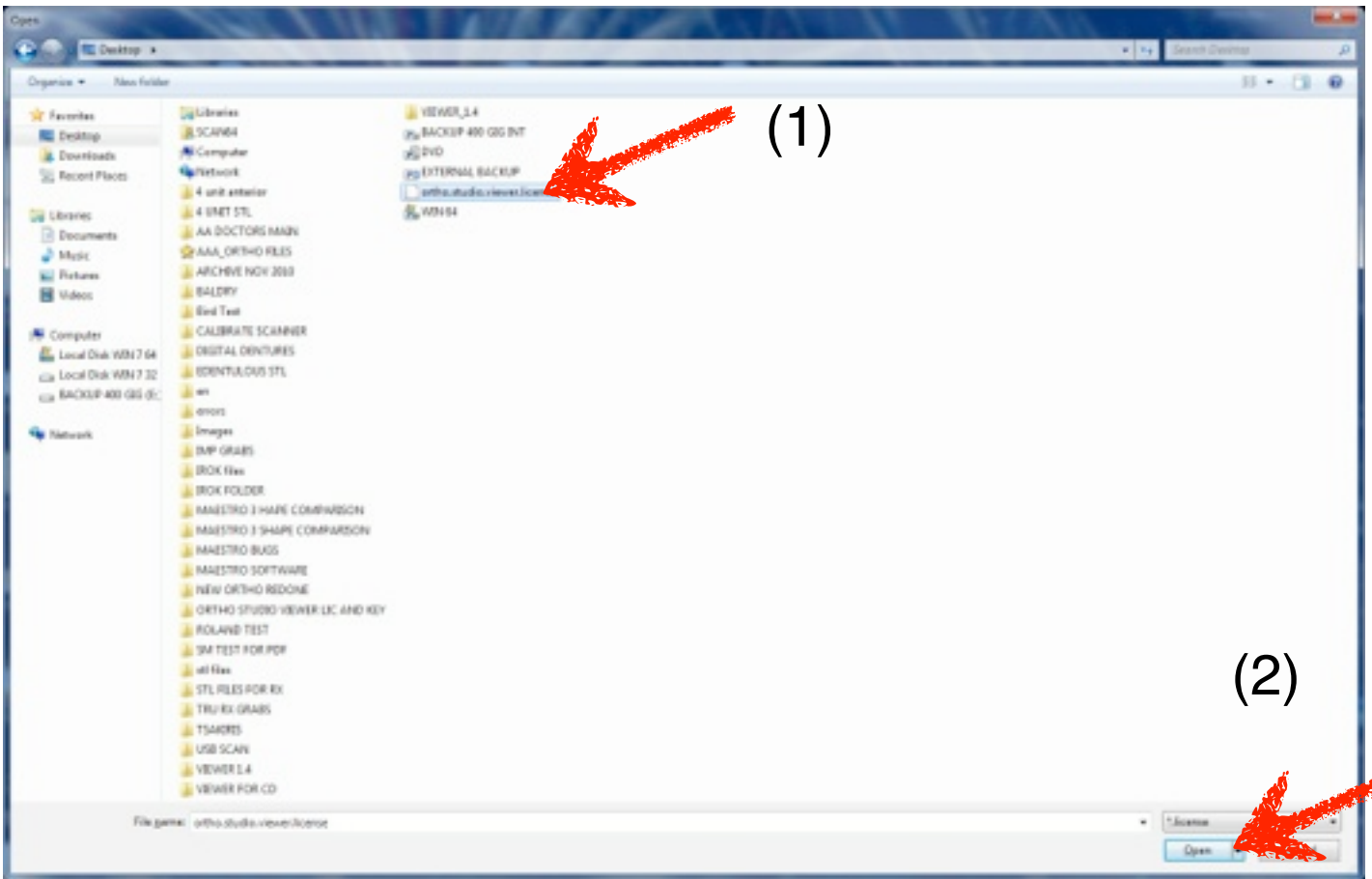

14. Select the license file (1) and click “Open” (2)

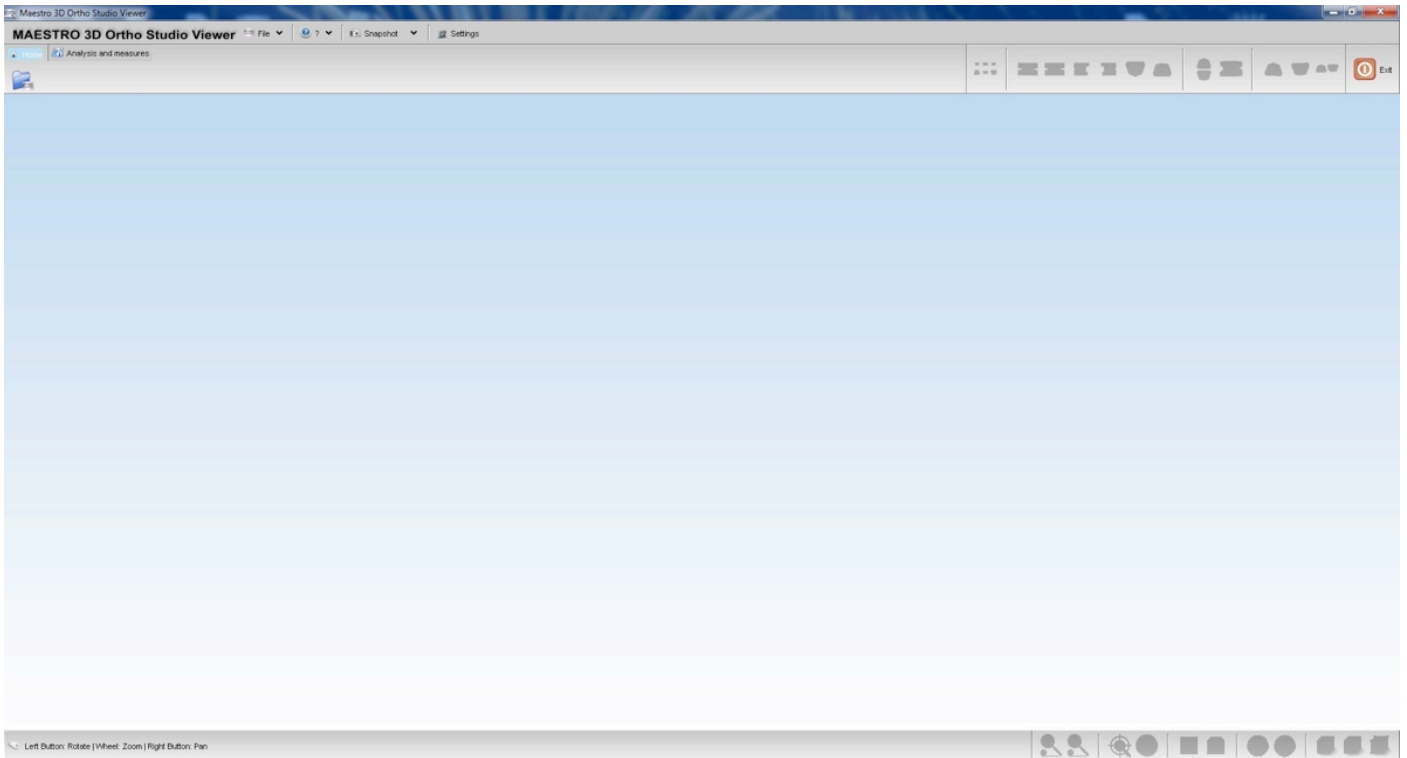

15. Maestro Ortho Studio Viewer will open automatically and will now be licensed.

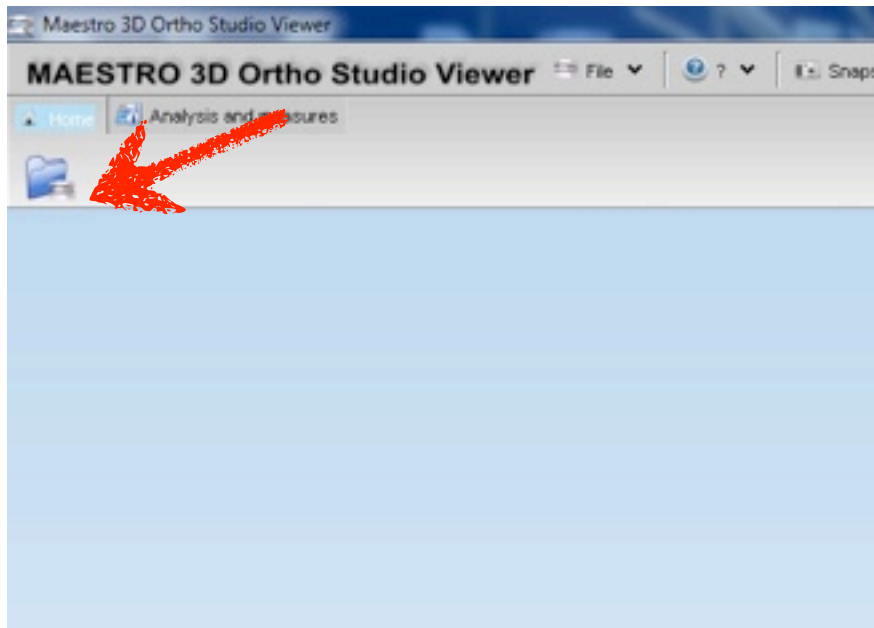

16. To get started click on this icon in the upper left corner to open a file

